
UK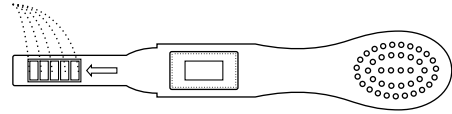

Ovulation Test Kit

Indicates fertile days

Please read this leaflet carefully before beginning your test.

When you are trying to become pregnant, it is important to understand when you are at your most fertile. Don't worry if you feel that it is taking longer to become pregnant than you anticipated, only 25% of couples will become pregnant in the first month of trying. Most couples take and average of 3 months to conceive and 10% take over a year.

One factor that can affect the time you take to conceive is timing and this is where ovulation tests can help. Typically there are only 2-3 days in your cycle when you can become pregnant. Alvita Ovulation Tests will indicate when these days occur by detecting the changes in your hormones during your menstrual cycle.

How the test works

Alvita Home Ovulation Test works by detecting the sudden increase in luteinising hormone (LH) in your urine that occurs 24-48 hours before an egg is released from the ovaries, a process known as ovulation. Once the surge has been detected you are most likely to become pregnant in this 48-hour period.

When should I use Alvita Home Ovulation Tests?

Firstly, you will need to work out which day to start testing. This will vary depending on the usual length of your cycle. The first day of your cycle (day 1) is the day that your period starts. The length of your cycle is the number of days from day 1 to the day before your next period. The diagram below enables you to work out the day to start testing from your cycle length. Work out the length of your cycle in one menstrual cycle. Then use the ovulation test in the next cycle.

Length of Cycle	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40
Start Testing	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

NB: If your cycle is irregular then use the shortest cycle you have had over the past 6 months to work out when to start testing e.g. if your cycle is 29 days long, then in your next menstrual cycle you should begin testing on day 13.

What time of day do I need to perform the test?

You may test at any time of day. However, you should not test with early morning urine, because although you can experience a surge in LH in the morning this may take up to 4 hours to show up in your urine. Reduce your fluid intake for 2 hours before testing as excessive liquid may dilute the hormone and make the test difficult to read. For the most accurate results you need to test at about the same time every day.

Performing the test

- When you are ready to begin testing remove the test from the foil wrapper.
- Remove the test's cap.
- With the tip pointing downwards, hold the absorbent sampler in your urine stream for at least 10 seconds, until thoroughly wet. Be careful not to splash above the arrow mark, as wetting the test window will affect the result. Still keeping the tip pointing downwards, remove the absorbent tip from the urine stream and replace the cap.
- Alternatively, you may wish to collect a sample of your urine in a clean, dry container. Immerse the absorbent tip only as far as the arrow, in the urine for 3 seconds. Keep the absorbent sampler pointing down for a few seconds and then replace the cap.
- Place the test on a flat surface with the window facing upwards and begin timing the test. The result is ready to read at 5 minutes regardless of which method you choose. Discard the test after 5 minutes.

Reading the results

- If there is no line in the test area of the window or the line in the test area of the window is lighter than the line in the control area of the window then your LH surge has not begun and you should continue testing daily.

- If the line in the test area of the window is the same or darker than the one in the control area of the window then the LH surge has been detected. You should ovulate in the next 24-48 hours.

- If there are no lines in either the control or test area of the window at 5 minutes the test has not worked. Also, if there is only a pink line in the test area of the window this again has not worked and is invalid.

The test stick should then be discarded and a new stick should be used for a further test.

Questions and Answers

My cycle length is longer than the range shown in the table. How can I find out when I should start testing?

Don't worry; begin by testing 17 days before you expect your next period.

What time of day should I test?

It is important to ensure that you test at approximately the same time every day. You should reduce your fluid intake for 2 hours before carrying out the test.

Do I need to use all of the test sticks?

No, once your LH surge has been detected you can stop testing and save the remaining tests to use next month if necessary.

I've used 7 tests, but I've not detected my surge. What should I do?

Don't panic, the number of tests in the pack are sufficient for most women with regular cycles. However, if you have a cycle length that varies by more than 3 days, you may have to use additional test sticks to detect your surge.

I've used the test, but no lines appeared. What should I do?

This may be because too little urine has been used. You should test again using a second test. Take care to ensure that the test is held in the urine stream for longer than 10 seconds.

Can the test results be interpreted after more than five minutes?

No, test results must be read at 5 minutes. Although a positive result should not change for several days, a negative result may change to a false positive within minutes after the end of the testing period, which would not be an accurate reading. It is always best to read the results at the 5 minute testing period and then discard the test to avoid confusion.

A pink background colour and vertical streaking appeared in the result area during the testing period. Is this a concern?

No, each urine sample will vary in its chemical makeup. Such variations can cause vertical streaking and/or the pink-rose background colour but will not affect the test results. As long as the control band appears within 5 minutes, the test is working properly.

What factors can affect the test?

If you are actually pregnant, have recently been pregnant, or have reached the menopause you may get a misleading result.

Fertility drugs containing luteinising hormone or human Chorionic Gonadotrophin may affect the results. It is advised that you check with your GP before using the tests. Clomiphene citrate (Clomid) should not affect the test, but can affect the length of your cycle causing you to require more than 7 tests. Breastfeeding can affect the test results.

I've recently stopped using the contraceptive pill, will this affect the results?

The contraceptive pill disrupts your natural hormone pattern, if you have recently stopped taking it, your cycles can be irregular and may take some time to stabilise. You may wish to wait until you have had two normal periods and your cycle has stabilised before testing.

Do I need to keep records of my daily readings?

No, each test will give a straightforward 'yes/no' answer.

Will a home ovulation test replace the basal body temperature method?

No, because it looks at ovulation from a different angle. If you wish you may use the two methods together to help you to calculate exactly when to start using the ovulation tests.

How long will the line remain visible?

The test should be read at 5 minutes for best results. A positive (Surge) result will never disappear. The coloured line(s) may become darker and a tinted background may appear after several hours. Some negative results may later display a faint second colour line because of evaporation from the test line. Therefore, you should not read the result after 5 minutes. Discard the test once you have read the result.

Once I see a positive result, when is the best time to have intercourse?

Ovulation is likely to occur within 24-48 hours. This is your most fertile time. Sexual intercourse within this time frame is advised.

Can I use the test to avoid pregnancy?

No, the test should not be used as a form of birth control.

Keep out of reach of children.

Store in a dry place at 4°C – 30°C.

Keep away from direct sunlight, moisture and heat.

Do not freeze

Do not use if foil wrapper is damaged.

Do not use a test stick that has passed its expiry date.

Each test stick can only be used once.

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