# HILARY® FRÜH

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# HILARY® FRÜH

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### Instructions

#### Simple to use, quick and reliable results.

With the HILARY® früh direct early pregnancy test, the pregnancy hormone hCG, which indicates pregnancy, can already be determined up to 3 days before the period is due. If the test is done before the period is due, morning urine should be used. After the period is due, the test can be used at any time of day. The test results will be ready after 5 minutes.

#### The test is very simple to do:

1. Remove the test rod from the foil. The packet in the bag contains a desiccant. It is not needed for the test.

2. Pull the cap off the rounded end of the test rod.

3. Hold the tip of the test rod into the ray of urine for 5 - 10 seconds until it is well moistened.

Or: You can also catch the urine in a clean, dry cup. Immerse the tip of the test rod in the urine sample for at least 15 seconds.






4. Then place the test rod on a surface with the results window up. You can read the results after 3-5 minutes.

**Results:**  
One red line in the small results window = control line  
**not pregnant**

Two red lines, one in the large results window and one in the small results window  
**pregnant**

The two lines may have varying color intensities, from red to weakly pink, and have different widths. This is normal. If the results strip is weakly pink, you should read the test again after another 15 minutes, since the strip may color more strongly in this time. If the test shows that you are pregnant, contact your physician to discuss the measures which are suitable for the course of the pregnancy.

If no line appears in the small window, the test did not work properly and the results are invalid.

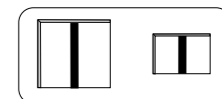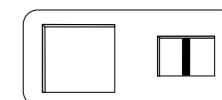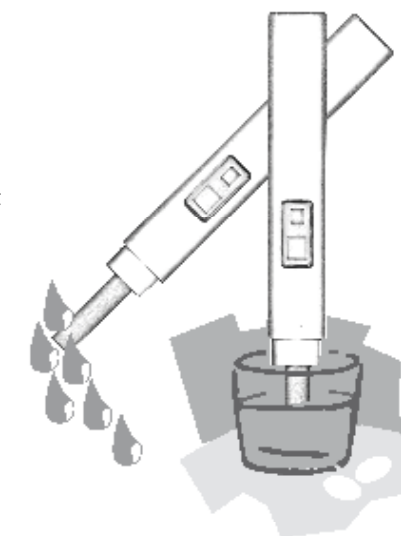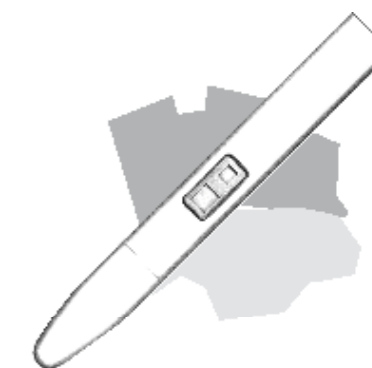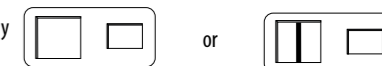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