Wundgel

D Gebrauchsinformationen

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VS 12/11

GB Instructions for use

For users: please read the following instructions for use carefully as they contain valuable information about application of pharmaCUR® Wound Gel.

pharmaCUR® Wound Gel**Characteristics**

pharmaCUR® Wound Gel is an odourless and colourless gel that has been developed especially to encourage healing of wounds. It keeps wounds moist, protects them from infection, actively promotes healing of wounds and reduces formation of scars. In addition, pharmaCUR® Wound Gel prevents adhesion of the wound to the dressing or plaster material.

Application of pharmaCUR® Wound Gel is pain free, irritant free and therefore is also suitable for children, people with allergies and diabetics. Application on mucous membranes has also proved successful.

The polyhexanide betaine complex contained in pharmaCUR® Wound Gel also decontaminates wound areas that are hard to reach, thus preventing infection of the wound.

The ingredients in pharmaCUR® Wound Gel have been used successfully in clinical practice, are safe for dermatological use and are considered the first choice in treatment of wounds.* pharmaCUR® Wound Gel has a very broad microbial spectrum (bacteria, viruses and fungi) and also provides protection against multiresistant pathogens (MRSA).

Areas of application

To encourage healing, prevent infection and regulate moisture in:

a) acute wounds: e.g. gashes, cuts, grazes, postoperative wounds; also on mucous membranes

b) chronic wounds: e.g. pressure ulcers and venous leg ulcers
c) wounds subsequent to 1st and 2nd degree burns

Application

A wound should first be cleansed of any dirt, blood, debris and germs. pharmaCUR® Wound Spray is especially suitable for this.

Then pharmaCUR® Wound Gel is applied to encourage healing of the wound.

The surface of the wound should always be covered generously with the gel. To do so, apply pharmaCUR® Wound Gel to the wound and then cover with dressing material. Compresses, gauze and plaster can also be wetted in order to cover wounds.

pharmaCUR® Wound Gel stays on the wound until the dressing is next changed and prevents the wound from adhering to the dressing material. When changing dressings, it is recommended you use pharmaCUR® Wound Spray to remove cell and skin residue that is stuck to the gel.

Unless otherwise instructed by the doctor or nurse, change the dressing once daily.

Continue application until the wound has closed up.

Restrictions for use

Do not use pharmaCUR® Wound Gel around the auditory canal or the eyes. Careful thought should be given to use of the product during pregnancy and while breastfeeding. If in doubt medical advice should be obtained. To date there is no evidence of harmful effects when used during pregnancy and while breastfeeding. As the ingredients are not absorbed into the system, it is not expected that they will be passed into the breast milk

General safety notices

pharmaCUR® Wound Gel should not be used in the case of allergies to one of the ingredients. For external use on wounds to the skin only, do not swallow. Keep out of reach of children! Seal tube after use.

Composition

Aqua purificata, glycerol, hydroxyethyl cellulose, 0.1% undecylenamidopropyl betaine, 0.1% polyaminopropyl biguanide (polyhexanide)

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