ProntoLind® Piercing & Tattoo Gel 10 ml

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ProntoLind® Piercing & Tattoo Gel 10 ml

ProntoLind® Gel is a ready-to-use gel for antiseptic care and prevention of infection in piercings and tattoos.

The ingredients have been used successfully in clinical practice millions of times over, are safe for dermatological use and are considered the means of first choice in treatment of wounds*. They do not inhibit granulation and epithelialisation, do not demonstrate any cytotoxic, inflammatory, irritating or sensitising properties and can be used safely over long periods of time.

The polyhexanide betaine complex contained in the ingredients also decontaminates areas of the skin, mucous membranes and jewellery that are hard to reach. Thanks to the resulting protective film and the broad microbial spectrum, it prevents new infectious agents from implanting themselves.

Dispersion of pathogenic, transient and resident bacteria in the piercing hole, around the jewellery and on the tattoo is thus prevented. This helps and accelerates the natural healing process in the wound.

In the case of flesh tunnels, ProntoLind® Gel prevents and reduces unpleasant odours to a minimum and can also be used as a lubricant gel for expansion (e.g. ear holes). ProntoLind® Gel is especially suitable for use with nasal and ear piercings, dermal anchors and microdermals, and for any areas

that are covered by a wound dressing. ProntoLind® Gel is compatible with all modern wound dressings and jewellery material.

Instructions for use: Use with existing piercings:

Apply a sufficient amount of ProntoLind® Gel directly to the treatment area and allow the gel to dry for a short time. When wound dressings are used, the gel can be applied directly to the compresses/plasters and thus additionally prevents adherence with the wound, which makes the next change of dressing easier and prevents fresh damage to the body jewellery. It is recommended that ProntoLind® Spray is used to clean the skin when changing the dressing.

Application when completing the tattoo: After completing a fresh tattoo, apply ProntoLind® Gel sparingly to the tattoo and rub it in carefully until it is almost dry. Thanks to the polyhexanide betaine complex contained in the gel, the skin is cleaned antiseptically and this keeps any risk of infection to a minimum. Afterwards cover the tattoo with foil for about 3 hours and continue to keep it cleaned until it has healed completely using ProntoLind® Foam for Tattoos.

Length and frequency of application: All ProntoLind® products can be used without restriction as regards the length and frequency of application, as they have only a positive effect on the healing of wounds.

Continue to observe the instructions provided by your piercer/tattooist and make sure you keep your appointment to have the wound checked – they know best about your body jewellery. **Safety notes:** Do not use ProntoLind® Gel at the same time as soap, ointments, oils or gels. May not be used on persons who are allergic to one of the ingredients.

Seal tube after use and protect from exposure to direct sunlight. Use within 12 weeks of opening. For external use only, do not swallow.

Composition: Aqua purificata, glycerine, hydroxyethylcellulose, polyaminopropyl biguanide (polyhexanide), undecylenamidopropyl betaine

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Hersteller & Vertrieb: Promted GmbH Am Bahndamm 70 D-32120 Hiddénhausen www.promted.de Tel.: ++49(0)5221-690011